

THALASLIM+

OPERATOR'S HANDBOOK

(IMPORTANT INSTRUCTIONS
TO KEEP FOR LATER USE)



: read the instructions



: do not use if folded or wrinkled



: do not insert pins

CE

DOC0397B

Congratulations! You have just acquired a **THALASLIM+ !** Read the following instructions carefully in order to get the most from your **THALASLIM+** device's possibilities.

After unpacking the equipment, inspect it. In particular, make sure that the device has not been visibly damaged due to shipping. If you have any doubts, do not use the device and contact your retailer. The packing, particularly the plastic bag, should be kept away from children because it is a source of danger.

This device is to be used for its intended application only: a thermal blanket for aesthetic purposes. All other use is considered improper and may be dangerous. The manufacturer is not responsible for improper or unreasonable use.

This device should be used on a covered premises; do not leave it out in the sun or rain.

In case of misuse or malfunction, turn off the device and contact your retailer: ignoring this recommendation will compromise the device's safety.


If the device is not operated for a certain period of time, it is advisable to turn it off. Do not leave the device turned on when not in use.

The device must be activated according to the manufacturer's instructions. Improper activation may damage persons, animals, or goods, for which the manufacturer cannot be held responsible.

1. DESCRIPTION

The **THALASLIM+** includes a **control box** and an electrical **heating blanket**.

1.1 Control box :

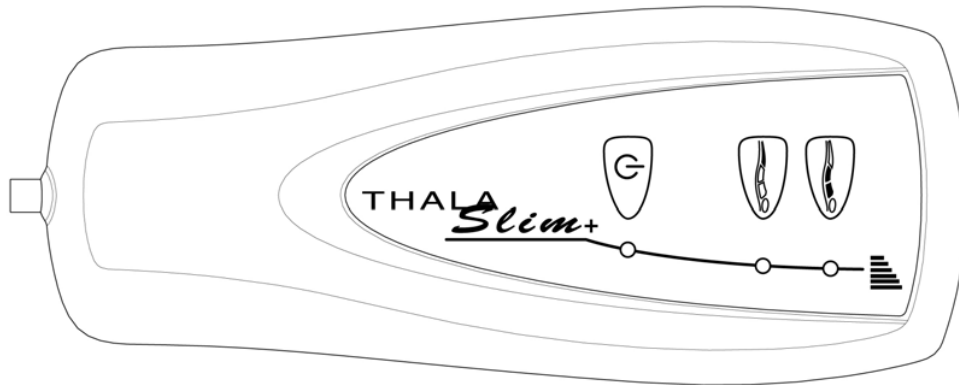


A full **treatment** includes **preheating time**, **duration of treatment** and **resting period**; every laboratory with a beauty treatment line has developed one or more beauty care protocols defining the durations of treatment and resting period : contact your cosmetic product laboratory to obtain this information.




With the 60' timer control box, you can programme separately:

- output of the large section (for bust and trunk),
- output of small section (for legs).

Key board of the control box has 3 keys:


-  key : treatment start,
- large section output key,
- small section output key.

Every key has a light :

-  key : as soon as supply cable is plugged, its light (on the right of the  key) flashes with a green light ; to start treatment, press the  key : green light becomes fixed ; 5' before the end of temporizing, green light flickers at first quickly, then more and more slowly as far as we almost reach the end of temporizing ; when green light flashes again, temporizing is over and output is off.
- output key lights to set up heating level may have different colours :
 - light off : heating off,
 - green colour : low level,
 - yellow colour : medium level,
 - red colour : high level.

When starting a treatment, levels displayed are those of previous treatment ; settings can be done only during treatment in process.

1.2 2 section heating blanket:

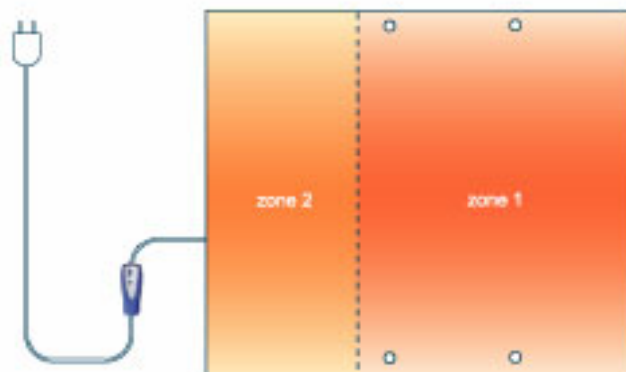


The thermal blanket should be used only with the control box described hereabove.

The two-colour (navy/silver) thermal blanket should be used with the navy side on the inside, and the silver side on the outside.


The thermal blanket has been "**Bio-Prüf**" treated and has a powerful anti-germ, anti-fungal agent incorporated into the plastic material; unlike surface treatments whose action is limited over time, "**Bio-Prüf**" acts throughout the device's entire lifetime by permanently diffusing active principles that prevent micro-organisms and fungus from attaching themselves to the device and reproducing, thus preserving all of the treated material's original characteristics. "**Bio-Prüf**" has bactericide action, avoids contamination and risks of infection by creating an "inhibition" zone; it is also a powerful anti-fungal element that protects the device from fungus, and eliminates allergy-causing elements and unpleasant odours.

The thermal blanket's shape, with a front closure, combined with update padding and foam back support to avoid marking the back, fits the body's shape perfectly: it has 2 heating sections with different sizes and outputs.

Before using the first time, spread the thermal blanket flat on the treatment table, folded in 3 as shown in the drawing below; after a preliminary 10-minute pre-heating, the plastic covering will smooth itself out.

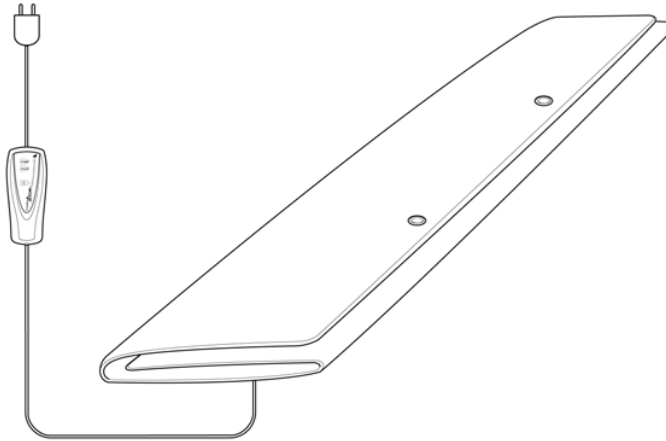

It is indispensable to use disposable plastic film (ref. FE160200) to avoid soiling the thermal blanket and to protect it from essential oils that may affect the exterior plastic covering's flexibility.

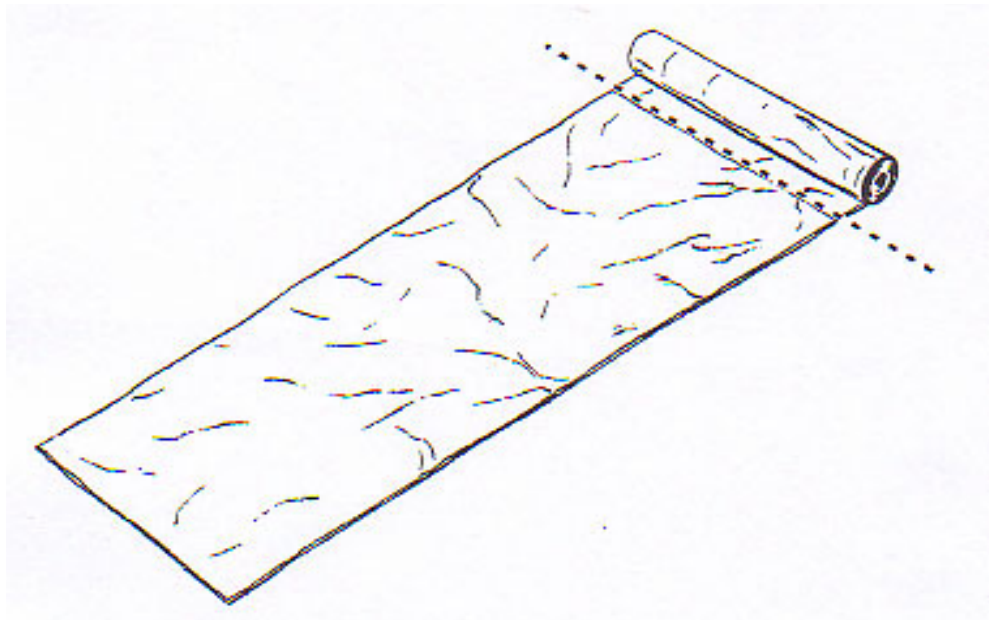


2. OPERATION

2.1 Before turning on for the first time:

Make sure that the control box's power supply voltage is the same as that one of the main supply.

2.2 Turning on the device:

Plug supply cable in the socket : green light of  key flashes slowly.

2.3 Turning off the device:

Unplug supply cable off the socket.

3. "THERMAL BLANKET" PREHEATING

Before starting treatment, preheat the thermal blanket for 10 minutes, folded in 3 as shown at § 1.2.

4. MODIFICATIONS DURING TREATMENT

While treatment in process, beautician can set section outputs by pressing output keys as many times as necessary to reach heating level requested :

- light off : heating off,
- green colour : low level,
- yellow colour : medium level,
- red colour : high level.

5. PRECAUTIONS FOR USE

- Do not use if wrinkled.
- Do not use if folded.
- Do not tuck in areas that contain the conductors.
- Do not insert pins.
- Do not use the device with persons who are weak, infants, or persons insensitive to heat.
- Do not use with self-heating cosmetics.
- Do not dry-clean.
- Do not place other objects on the thermal blanket.
- Do not use the cable to pull the device.
- Do not use on an adjustable bed, or if used on an adjustable bed, check that the blanket or cord cannot become trapped or rucked.
- Do not use arms outside with this blanket; to get arms outside, use FREESLIM blanket.
- This blanket is not to be used by young children unless the controls have been preset by a parent or guardian or unless the child has been adequately instructed on how to operate the controls safely.
- Examine the device frequently to check for signs of wrinkling or damage. If such defects are visible, or if the device has been used in an abnormal manner, return it to the supplier before any further use. The covering can be used intensively for approximately 2 years; it is then necessary to replace the heating section with heating section spare.
- When the device is not in use, fold and store flat. Do not place anything on top of the device.
- The covering can be cleaned with a soft sponge and soapy water. Make sure that the power supply cable is unplugged before cleaning. Do not clean with solvents. **Do not turn on the device until it has dried.**
- This device is not designed for use in hospitals.
- Advise the patient to ask his or her physician's advice before undergoing any treatment.

- If the power supply cable is damaged, it must be replaced by the manufacturer, after-sales service, or a similar qualified individual in order to avoid danger.

6. MAINTENANCE

6.1 Unplug main cable before carrying out any maintenance operation.

6.2 Control box :

- clean with a damp cloth, do not let any liquid run into the programmer,
- do not use solvents
- surface damage due to scratches, impact, or chemical damage to the paint is not included in the warranty


6.3 Thermal blanket :

The thermal blanket is manufactured with "Bio-Prüf" treated material, which guarantees permanent hygiene against all germs and fungus; the thermal blanket therefore does not need to be disinfected, but simply cleaned between each treatment as follows:

- dust with a damp cloth
- remove stains with a cloth soaked in soapy water (do not use solvents).
- rinse with a sponge and water

7. TROUBLESHOOTING

7.1 The control box does not flash when the device is turned on:

- Make sure that the socket has correct power supply by plugging in another device.
- If there is correct power supply to the socket and the  key light does not flash, return the Bodyslim to your distributor.

7.2 A section does not heat, but its light is on :

Contact your distributor to repair the thermal blanket.

8. TECHNICAL CHARACTERISTICS

- dimensions : 1.54 x 1.70m
- maximum output : 250W
- power supply voltage: 115 Volts AC, 230 Volts AC
- standard colour: navy / silver
- heating zones: 2
- class II device
- standards: EN 60335-1 & EN 60335-2-17

9 TERMS OF WARRANTY

Our supplies are guaranteed for one year against all manufacturing defects, starting from the date that they are in the buyer's possession (the Thalasslim+'s serial number should be included in the retailer's sales invoice). Our warranty can not take effect until we have examined the material deemed to be defective by the buyer and returned at his or her expense to our Headquarters, following our company's written consent.

Our manufacturer's warranty is strictly limited to the repair or replacement of our supplied product: our Company cannot be held responsible in any case for loss or damage, direct or indirect, to the buyer or third parties caused by the product.

Repair, modification, or replacement of parts during the warranty period will not prolong the equipment warranty deadline.

The warranty will cease without further consideration if the buyer has attempted repair or modification at his or her initiative.

There is no warranty for second hand equipment.

There is no warranty for equipment that is not identified by its serial number located on the label delivered with the device; this serial number must imperatively appear on the user's invoice.

The contractual warranty above in no case hinders the application of the legal warranty.