

Benefect® Botanical Decon 30 Disinfectant

*This product is proven to kill over 99.99% of the designated test microorganisms required for broad-spectrum claims of hospital disinfectant effectiveness: Pseudomonas aeruginosa, Salmonella enterica, Staphylococcus aureus, Methicillin Resistant S. aureus (MRSA), Escherichia coli, Influenza A virus, H1N1 Influenza A virus, Influenza A Avian (H3N2) virus, Rhinovirus type 37, Norovirus, Human Immunodeficiency Virus (HIV) Type 1, Trichophyton mentagrophytes, Aspergillus niger & Candida albicans. Kills over 99.99% of bacteria in just 30 seconds.

Clean And Disinfect In One Easy Step.
Prevents cross contamination. Prevents Mold & Mildew.

SUITABLE FOR use on all hard, non-porous surfaces in restoration, decontamination & remediation sites (fire, water, sewage & mold) & residential, medical & commercial applications including: health care facilities, food preparation facilities, hospitals, nursing homes, medical, veterinary & dental offices, schools, daycare facilities, health professional, chiropractic & physiotherapy clinics, nurseries, sickrooms, bathrooms, basements, kitchens, restaurants, food storage areas, offices, fitness gyms, spas, hotels, zoos, barns, kennels, planes, trains, automobiles & ambulances.

Effectively eliminates odors produced by odor-causing bacteria. Effectively deodorizes odors on concrete, wood framing & studding, trim, bedding, upholstery furniture, carpet, pads & subfloors. Provides residual deodorizing activity. No harsh fumes.

SUITABLE FOR disinfecting hard, non-porous surfaces (metals, glass, fiberglass, plastic, vinyl and similar polymers, glazed tiles, painted or sealed surfaces, finished woodwork, laminated surfaces, sealed marble, and similar sealed stone or stone-like composites) including: household contents, remediation equipment & PPE (Personal Protective Equipment), floors, vertical surfaces (sanitize in 30 seconds), walls, construction materials such as water damaged or sewage contaminated sheet metal, sealed wallboard or wood, shoes & boot soles, coils & drain pans of HVAC & R equipment, food preparation surfaces, cutting boards, countertops, tables, sinks, toilet seats, fixtures, cabinets, door knobs & handles, bed frames & rails, computer keyboards & mice, garbage cans, children's toys, highchairs, changing tables, pet areas, sports equipment such as jock cups & helmets. This product is not for use on medical device surfaces.



Proprietary Formula
Distributed By: Sensible Life Products
555 Bay St. North
Ontario, CA L8L1H1
www.Benefect.com
(905) 528-7474
Country of Origin: USA
ESL: 072318 REV: 082318



Benefect®



Kills Over 99.99% of Bacteria in 30 Seconds!

Fungicidal • Virucidal* • Bactericidal • Mildewcidal • Antimicrobial Germicide

HOSPITAL DISINFECTANT CLEANER • DEODORIZER • DECONTAMINANT • CARPET SANITIZER

ACTIVE INGREDIENT:

Thymol* 0.05%
*(present as a component of botanical essential oils)

OTHER INGREDIENTS: 99.95%
TOTAL: 100.00%

Light Lemon & Thyme Scent

Ready To Use

Keep Out of the Reach of Children

Net Contents: 1 Gallon (3.78 L)

A proprietary formulation made from botanical essential oils.
Part of our botanical cleaning & disinfection solution.

DIRECTIONS FOR USE: It is a violation of Federal law to use this product in a manner inconsistent with its labeling. For heavily soiled or greasy areas, pre-cleaning is required.

TO DISINFECT: 1. Wet the surface by sprayer (high or low pressure, hand or power operated), cloth, sponge or mop (spot test to check surface compatibility). 2. Leave for 10 minutes. Allow to air-dry. **No rinsing or wiping is required (even on food contact surfaces).**

TO SANITIZE: 1. Wet the surface (spot test to check surface compatibility). 2. Leave for 30 seconds. Allow to air-dry. **No rinsing or wiping is required (even on food contact surfaces).**

TO DEODORIZE: 1. Wet the surface (spot test to check surface compatibility). 2. Allow to air-dry. **No rinsing or wiping is required (even on food contact surfaces).**

TO CLEAN: 1. Wet the surface (spot test to check surface compatibility). For stubborn stains, let stand for a few minutes.

TO SANITIZE CARPETS: Apply to the pre-cleaned carpet area by wand, sprayer, extractor, spin bonnet or immersion processes until saturated (spot test fibers for colorfastness before use). Scrub or agitate the carpet as needed to ensure the test substance is delivered throughout the carpet pile and backing. Allow the test substance to remain on the carpet undisturbed for a minimum of 60 minutes. Extract & allow to air-dry.

FOGGING INSTRUCTIONS: For ULV fogging in contained vacant areas as part of pre-cleaning or contamination control (knock-down) in restoration or remediation sites, as a supplement to final cleaning and disinfection. This is a ready-to-use product; do not dilute. After setting up fogging device, seal all windows & doors to confine the fog & leave area during the treatment time recommended by the device manufacturer. After fogging cycle is complete, fully ventilate the treated area until suspended product is purged prior to re-entry. Do not enter the fogged area until the fog has completely settled or exhausted. Do not use equipment in the fogged area until the treatment has dried. After fogging pre-treatment, follow the product label's cleaning & disinfection directions.

KILLS HIV ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with body fluids and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficiency virus Type 1 (HIV-1) (associated with AIDS). **SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 ON SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS.** Personal Protection: Specific barrier protection items to be used when handling items soiled with blood or body fluids, are disposable latex gloves, gowns, masks and eye coverings. Cleaning Procedure: Blood & other body fluids must be thoroughly cleaned from surfaces & objects before application of the disinfectant. Disposal of Infectious Material: Blood & other body fluids must be autoclaved & disposed of according to federal, state & local regulations for infectious waste disposal. Contact Time: Leave surface wet for 10 minutes.

STORAGE/DISPOSAL: Non-refillable container. Store airtight at room temperature. Do not reuse or refill this empty container. Offer empty container for recycling. If recycling is not available, discard in trash.

EPA Reg. No 84683-3-74771

EPA Est. No 075840-CAN-001