



EarthLite®

WORLD'S #1 BRAND IN MASSAGE™



Vibra-Therm™ Sports Therapy Table

Thank you for purchasing the Earthlite® Vibra-Therm™ Sports Therapy Table!

Enjoy the convenience of a portable table paired with the luxury of an integrated relaxing heat and massage system. This user guide will walk you through instructions for using the Earthlite heat and massage system controls to maximize your client's comfort and relaxation.

The table should not be used in any other manner than described in this manual. Please read, understand, and follow all safety warnings, labels, and instructions. Misuse, use of unnecessary force, or using the product for unintended or unauthorized procedures can result in injury to operator, client, and/or equipment.

If you have further questions, please contact us at (800) 872-0560 for US customers, or (760) 599-1112 for international customers. We can also be reached via e-mail at info@earthlite.com.

NOTE: For instructions on how to set up and adjust the height of your table, see the Portable Massage Table Setup Guide. This manual only contains instructions for the heat and massage features of your table.

IMPORTANT SAFETY INFORMATION



To reduce risk of electric shock:

1. Always unplug table immediately after using and before cleaning or servicing.
2. Do not reach for controller that has fallen into water. Unplug immediately.



To reduce risk of burns, fire, electric shock, or injury to persons:

1. Do not leave table unattended while plugged in. Unplug when not in use.
2. Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
3. Never operate the table if it has a damaged cord or plug, or if it is not working properly. If it has been dropped or damaged, or dropped into water, contact Earthlite customer service.
4. Close supervision is necessary when this table is used by or near children or disabled persons.
5. Keep the cord away from heated surfaces.
6. Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
7. If you or client is subject to any medical condition involving the use of medical device products, and/or if they are under a physician's care, please consult with physician before use of heat/massage.

HAND CONTROL INSTRUCTIONS

Your hand control is already connected and ready for use. See below for a description of each button's functionality.

POWER

Activates Massage System

- Press once to turn on and set 15 minute timer (green light)
- Press and hold to set timer to 30 minutes (amber light)
- Press and hold again to turn off

Note: Unit may also be turned on by pressing any Zone 1-3 keys.

ZONES

(Green light indicates active Zone)

- 1 - Upper Back
- 2 - Lower Back
- 3 - Hip Region

INTENSITY

Controls intensity during all modes.

- Press (+) to increase
- Press (-) to decrease

SPEED

Controls pulsing frequency of massage motors in PULSE mode and controls the velocity as it cycles through in WAVE mode.

- Press (+) to increase
- Press (-) to decrease

HEAT

Activates Heat Element.

- Press LO (amber light) to turn on low heat (30 min timer)
- Press HI (red light) to turn on high heat (30 min timer)

Note: Works with massage or independently.

MODES

(Green light indicates active mode)

ZONE

Lets you choose any or all zones for massage.

- Press SELECT
- Press any or all Zones (1-3) to activate
- Press any or all Zones (1-3) to cancel

WAVE


Massage motors move up and down in a wave-like action

- Press WAVE
- Press any or all Zones (1-3) to activate
- Press any or all Zones (1-3) to cancel

PULSE

Massage motors turn on and off to simulate a pulsing action

- Press PULSE
- Press any or all Zones (1-3) to activate
- Press any or all Zones (1-3) to cancel

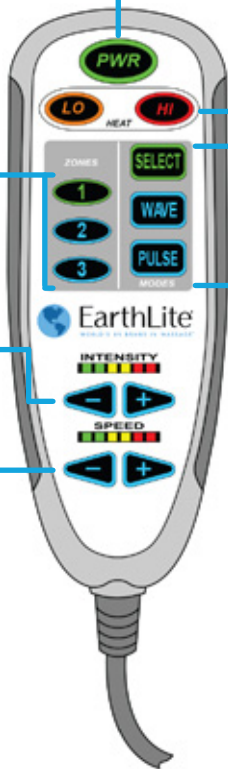

Contact Us



EarthLite[®]

WORLD'S #1 BRAND IN MASSAGE[™]

990 Joshua Way
Vista, CA. 92081
(800) 872-0560
(760) 599-1112 Int'l
(760) 477-6071 fax
info@earthlite.com

Hours of operation:

Monday - Friday
8:00 a.m. to 5 p.m. PST

Saturday - Sunday
Closed

Visit us online at:
earthlite.com



Join us on
Facebook



Follow us
on Twitter