



POSTURAL SPIDER

 Partner Application

ONE PIECE PRECUT
READY-TO-APPLY

Application Tips:



Ensure skin is dry and free from oils, lotions, and hair



Do not touch adhesive with your fingers



Do not stretch the tape when applying



Gently rub the tape after applying to activate the adhesive


Following these tips will ensure maximum benefits and extend the use of your product. Enjoy!

POSTURAL SPIDER


Application instructions

Application Tips:

- Notice the tab numbers on the back side of the tape showing the order of application.
- Before you begin, bend and snap all the perforations.
- Ensure skin is clean, dry, & free from oils before application.

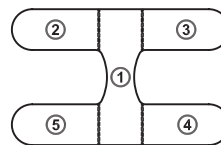

 Partner Application

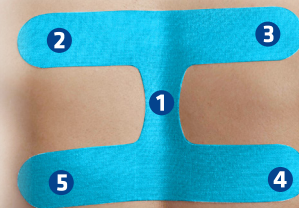

1



Have the user stand in a relaxed position.



Remove the number 1 tab and apply the section along their spine between their shoulder blades. Rub tape to activate the adhesive.

3



Take your time and gently rub the tape from the middle out to the ends to activate the adhesive.

2



Have the user rotate their palms facing forward.



Remove the number 2 tab and apply the section to their skin.



Maintain the same palms forward position, repeat this process with the number 3, 4, and 5 tabs.



Watch our
how to apply
video here