

# CLEANING AND DISINFECTING

*Remove day-to-day dirt with a mild soap and warm water solution, and gently wipe the surface dry. Please follow the use instructions below to maximize the life of your upholstery. For disinfecting or tougher stains, we recommend a 1:10 bleach/water-based solution or one of the cleaning and disinfecting products listed below. Avoid saturating or soaking the fabric with cleaner/disinfectant. Rinse with clean water after exposure to eliminate residue and gently dry with a soft cloth. Following these guidelines will extend the life of threads, seams, and cushioning that may potentially be impacted by disinfectant residue. Please use all cleaning and disinfecting agents as instructed. The use of other cleaning agents, disinfectants, conditioners, or protectants such as Armor All®, acetones, or citrus based solvents can degrade fabric performance and may void Earthlite's warranty. For intensive healthcare applications, we recommend considering our PVC or Ultraleather™ upholstery options for greater fabric life expectancy.*

*Extended and sustained exposure to direct sunlight, extreme temperatures, and damp/highly humid environments will shorten fabric life and may void Earthlite warranty. Avoid exposure to adhesives and abrasive materials.*

## **Bleach based Disinfectants**

- 1:10 bleach/water solution
- Clorox® Dispatch™
- Clorox® Germicidal Bleach

## **Alcohol Based Disinfectants**

- Isopropyl/Rubbing alcohol (70%)
- Veridien Viraguard®
- Fantastik®
- Formula 409®
- Clorox® Disinfecting Wipes
- Lysol® Disinfecting Wipes
- Metrex® CaviCide™
- Metrex® CaviWipes™

## **Hydrogen Peroxide Based Disinfectants**

- Hydrogen Peroxide
- Ecolab® Oxycide™